


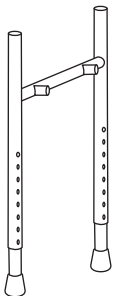
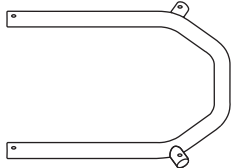
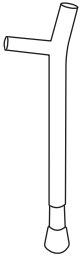
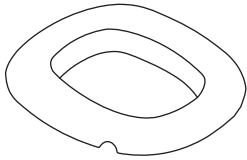
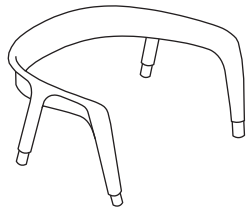
Let's Help Me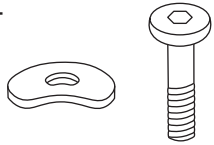


x 1 	x 1 	x 2 
x 1 	x 1 	x 4  x 1 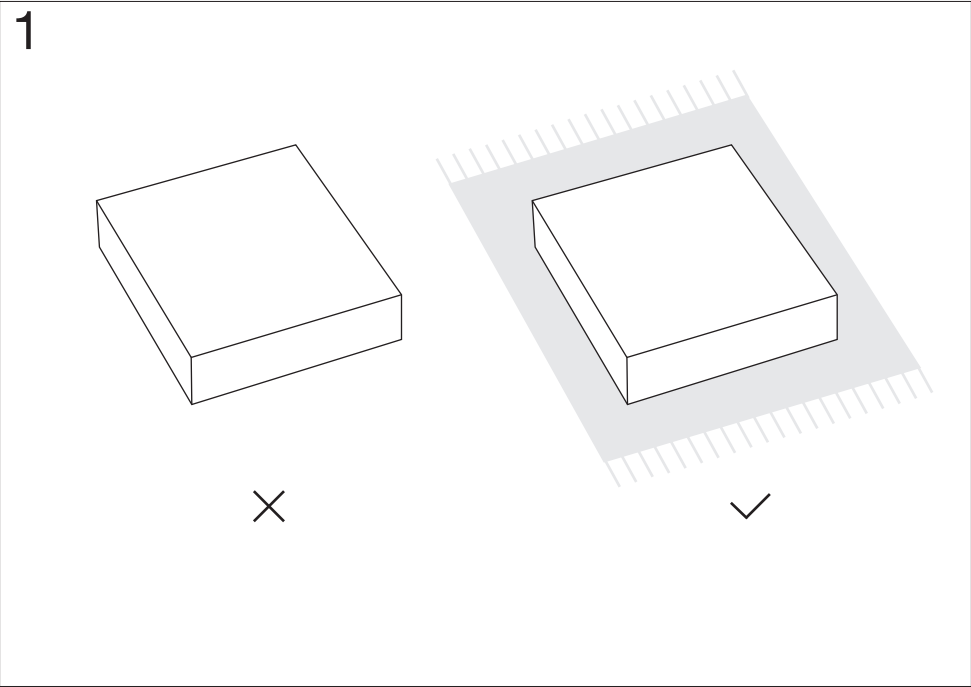


2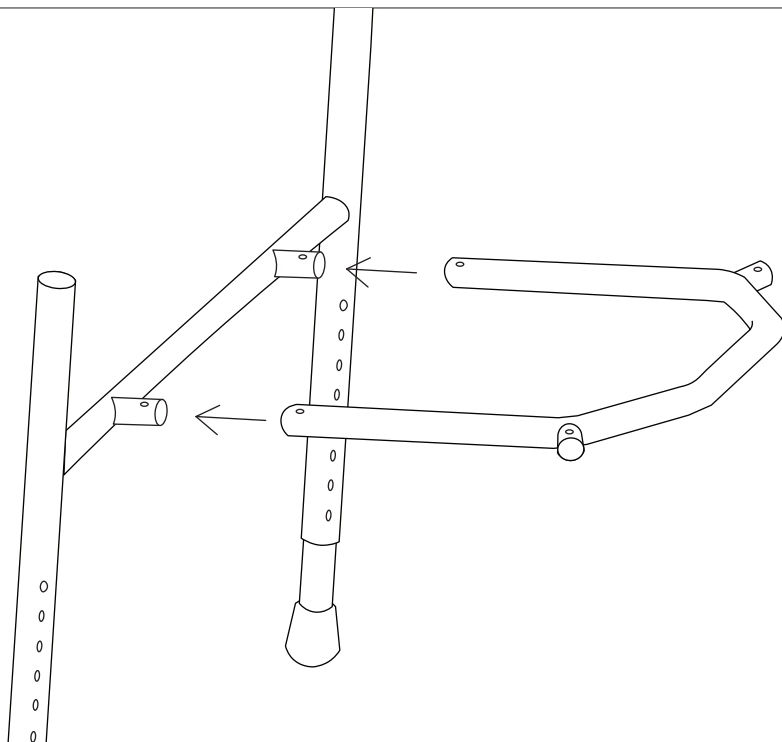


3




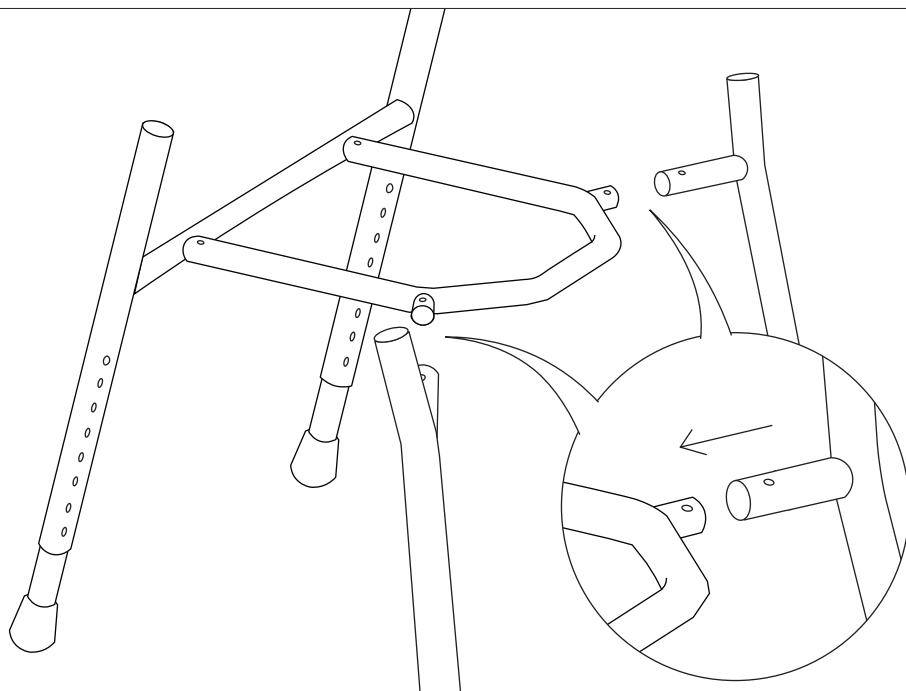
4

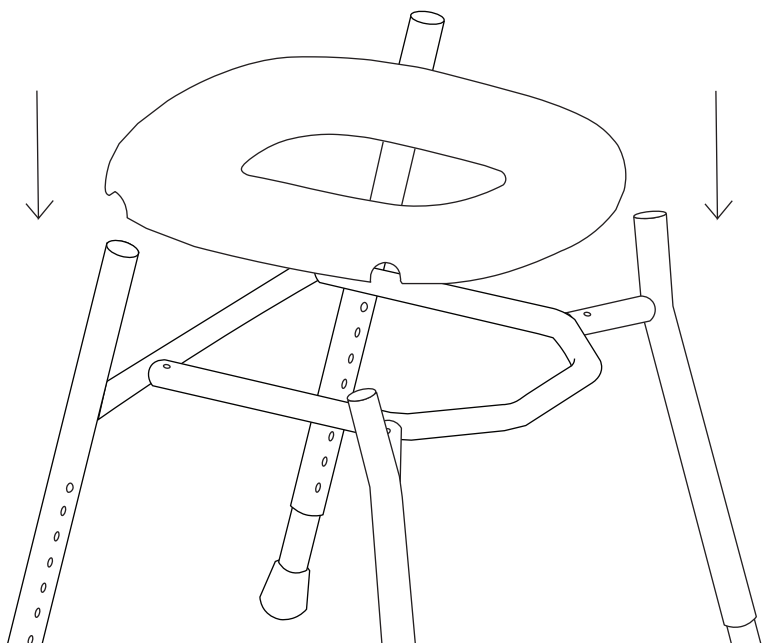
5

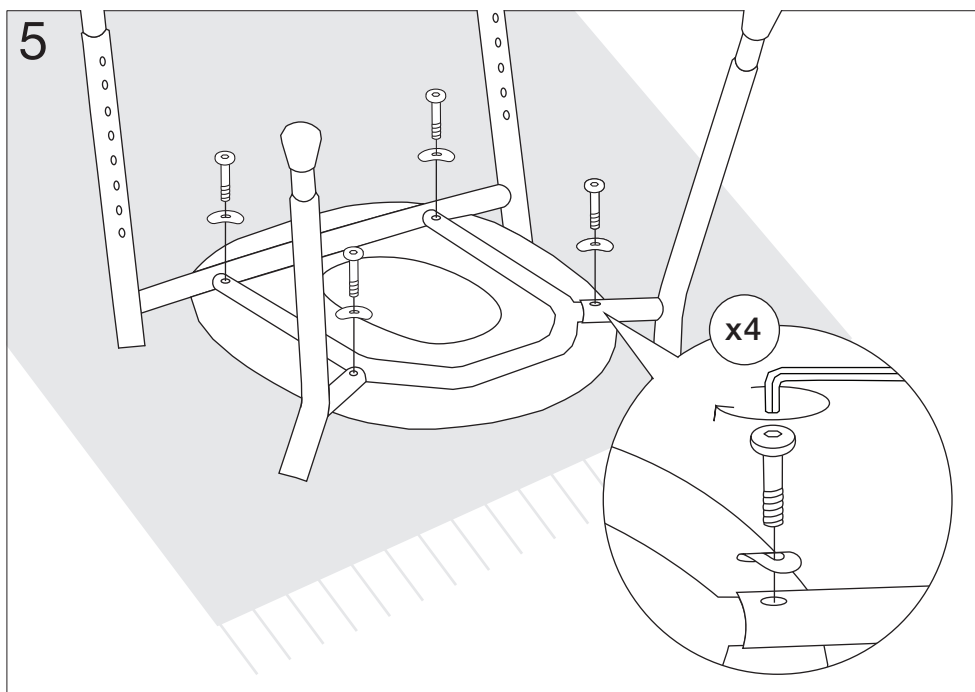
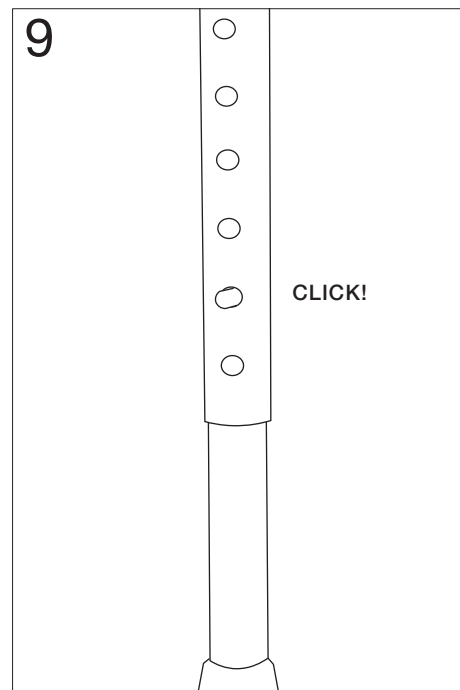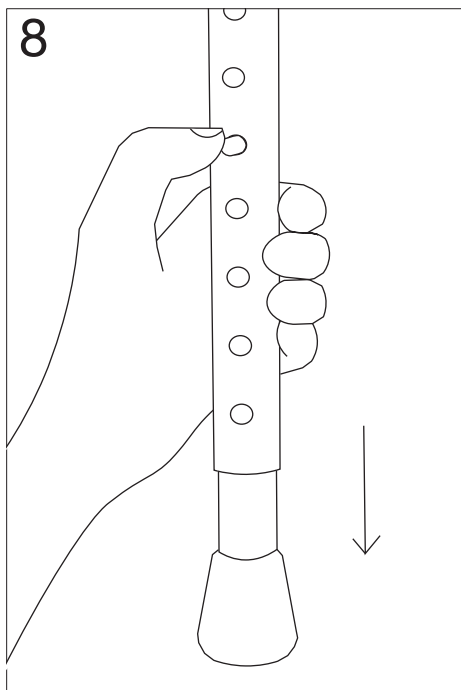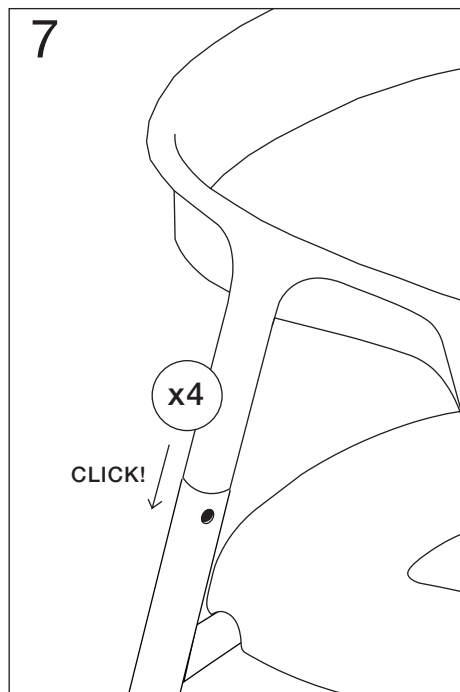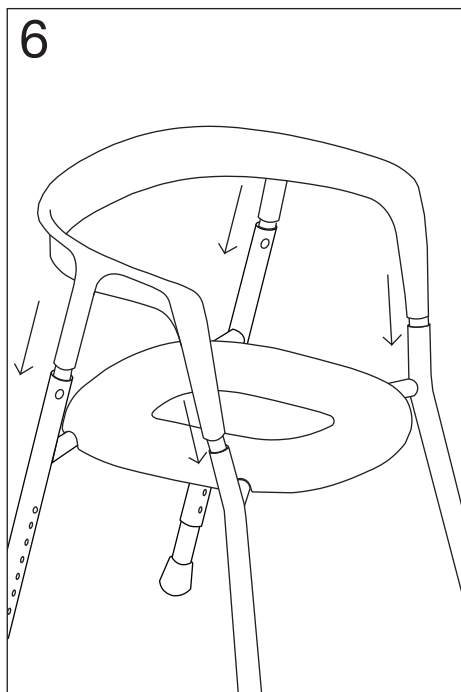
10

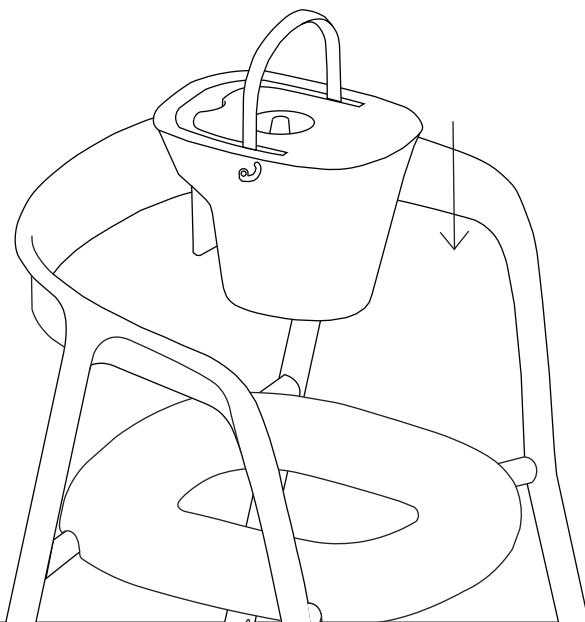
11

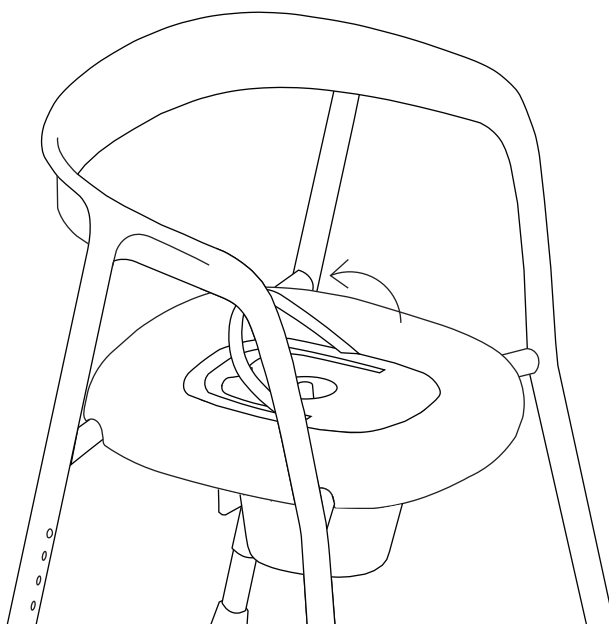12

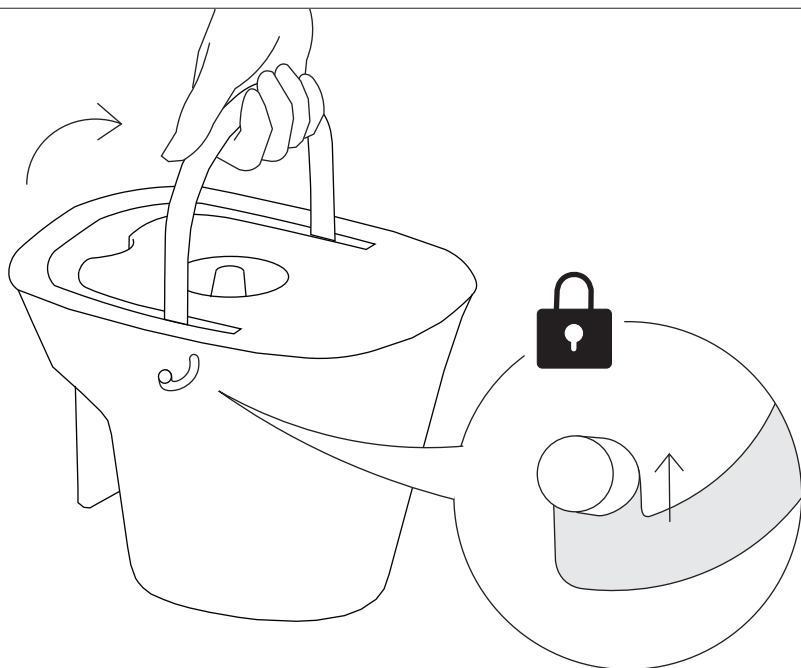13

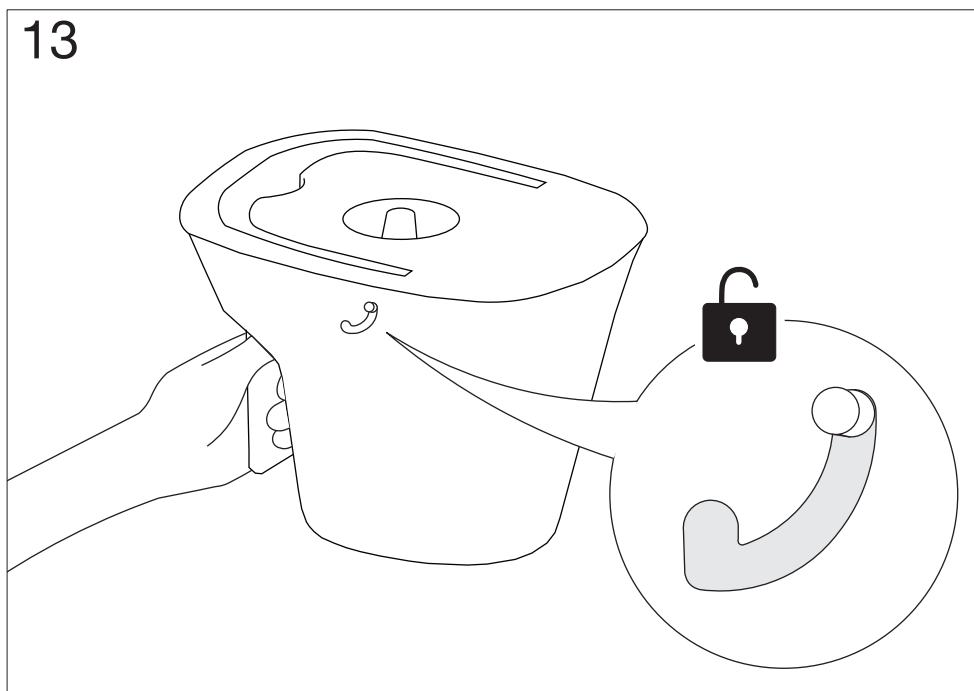Let's Help Me. Toalettförhöjning.

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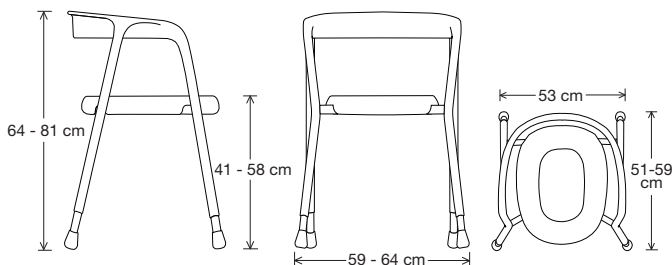
Let's Help Me. Commode.

Trust Care congratulates you on your new modern designed commode. Please note, it's important to read the manual before use.

Purpose & Use; A commode is the perfect choice for those of you who want to be able to get up more smoothly from the toilet seat, with help from Let's help Me, you can now easily set the desired seat height when visiting the toilet according to your needs. Not only this, you can also use the tub, which is easily placed (see pictures 10 & 11), to use the product as a free-standing toilet. When emptying, you simply put the lid on, which you then lift by the handle, which then locks automatically (see picture 12).

With the telescopic legs, you can easily set the desired height and with the anti-slip feet made of rubber, you can place the product on any surface to get a stable attachment (See also pictures 8 & 9). Back & armrests have a comfortable and ergonomic design that facilitates getting up and down, as this product is also water resistant, you can also use it as a stool in the bathroom or in the shower.

NOTE: The product is NOT intended to stand on.

Safety instructions: The product must not be used in any way other than what it is intended for. Trust Care is not responsible if the product is used, assembled, or modified in a way other than what is stated in this description. Regularly check the stability and strength of the product and that screws are properly tightened during assembly.

Maintenance & care; For hygiene reasons, it is recommended to clean the product regularly. Solvent-free cleaners with a pH of 5-9 or 70% rubbing alcohol are recommended. If necessary, the product can be decontaminated at a maximum of 85 degrees for approx. 3 min.

Guarantee: Trust Care offers a one-year warranty, however, the warranty does not cover normal wear and tear, e.g. of rubber feet.

Technical Data

CE

Tested according to: ISO 17966:2016, EN ISO 10993-1 for a maximum user weight of 160 kg.

Height	64 - 81 cm
Seat height	41 - 58 cm
Depth	51 - 59 cm
Width	59 - 64 cm
Weight	3.1 kg

The technical drawings illustrate the commode from three perspectives: side, front, and top. The side view shows the telescopic legs with a height range of 64-81 cm and the seat height range of 41-58 cm. The front view shows the seat height range of 41-58 cm and the base width of 59-64 cm. The top view shows the seat width of 53 cm and the depth of the seat area as 51-59 cm.

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